

I N S T R U C T I O N S




Hinged Elevated Toilet Seat

Catalog Number 72571 Series

U.S. Patent #5,251,338

Other Patents and Int'l. Patents Pending

1. Remove existing seat and lid from the toilet.
2. Place the two rubber washers (C) over the two bolt holes of the toilet.
3. Place Hinged Toilet Seat Unit (A & B) on toilet, small slotted holes up, making sure the holes line up with the holes in the rubber washers (C).
4. Place existing toilet seat over the hinged toilet seat.
5. Insert the bolt (D) through the toilet seat hinge, the mounting block of the hinged seat (B), the rubber washer (C) and the toilet.
6. Repeat with the second bolt (D) in the other hole.
7. Slip washer (E) onto the end of the bolt (D) and tighten nut (F). Repeat with second bolt.

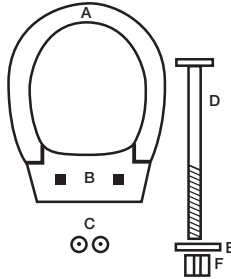

- Parts
- A. Lifting Riser
 - B. Mounting Block
 - C. Rubber Washers
 - D. Seat Bolt
 - E. Washer
 - F. Nut



WARNING:
HINGED TOILET SEAT SHOULD NEVER BE USED WITHOUT THE NORMAL TOILET SEAT ATTACHED AND HINGED IN THE DOWN POSITION

Maddak

661 ROUTE 32 SOUTH • WAYNE, NJ 07470-6814 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com