

HOW ARE LIFTING SEATS DIFFERENT?

Lifting Seats are unique products that help you get up from your chair. Available in portable or chair models, they use patented **LevelLift™ Technology*** that gently curves the seat to lift you safely, on a level plane without pushing you forward.

Lifting Seats are best suited for those with light to moderate difficulty standing up on their own from a seated position, often due to arthritis, neuromuscular, diabetic, cardiovascular or orthopedic conditions.

• Portable Self-powered Models

- Safe lift assistance of up to 70% of your bodyweight up to 350 lbs.
- Can be used on most armchairs or sofas
- Adjustable weight ranges for custom support
- For indoor or outdoor use, since they use no batteries or electricity

• Portable Electric Models

- Gentle, 100% lift for those up to 300 lbs.
- Can be used on most armchairs or sofas
- Easy-to-operate and can be stopped at any height
- Perfect around the home or when visiting
- Awarded the **Arthritis Foundation Ease-of-Use Commendation**

• Full Chair Electric Models

- Safe 100% lift for those up to 300 lbs.
- Discreet and stylish design
- Compact size for smaller areas, corners or next to walls
- Awarded the **Arthritis Foundation Ease-of-Use Commendation**


* Patent Numbers: 5,316,370 / 6,702,383

WHY CHOOSE LIFTING SEATS?

Lifting Seats help you stay active and independent by providing assistance **when** and **where** it's needed. Lifting Seats are the safer alternative to struggling to get up. Help reduce stress on your muscles and joints to save energy for other activities.



STAY ACTIVE AND INDEPENDENT

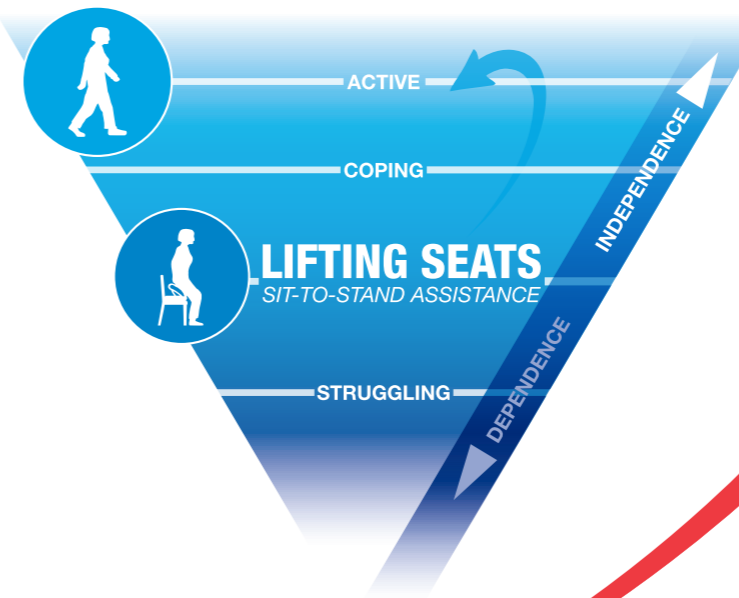

WHAT ARE LIFTING SEATS?


Lifting Seats keep you active and independent. Electrically powered and self-powered versions provide the right level of assistance based on your individual needs.



WHO CAN BENEFIT FROM LIFTING SEATS?

Those who are able to safely walk on their own, with or without a walking aid, and once standing are active at home or in the community.

"The Seat Assist has re-opened doors we thought had closed. You never think something as simple as standing up can change what you can enjoy and do in a day."
- Joseph & Ann

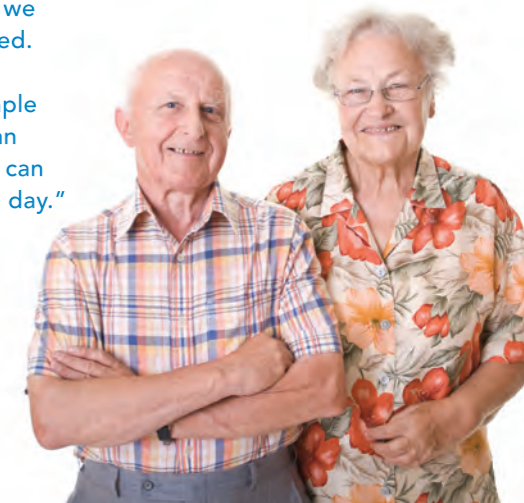

Carex Health Brands
800.526.8051
carex.com




WWW.CAREX.COM



Carex Health Brands



"Not needing to rely on my wife to help me out of my chair means I can enjoy my home and hobbies again."
- Mark

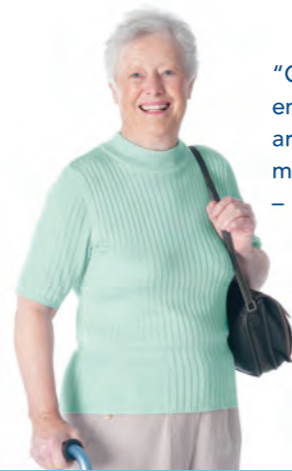

"Getting up and having enough energy to get around is everything to me; it is my independence."
- Ellen



"My Risedale fits perfectly with my home decor and my lifestyle."
- Edith



CHOOSE YOUR LIFTING SEAT

SEAT ASSIST

Up to 70% Sit-to-Stand Assistance

PREMIUM

STANDARD

POWER SEAT

100% Sit-to-Stand Assistance

PREMIUM

STANDARD

Risedale
Up in Style™

100% Sit-to-Stand

| MODEL(S) | MEDUL100 MEDUL300 | UPE1 UPE3 | PS3017 PS3020 | UPEP100 | RD03 |
|-----------------|---|---|--|------------------------------------|---|
| POWER SOURCE | SELF-POWERED | SELF-POWERED | ELECTRIC (120VAC 60HZ 0.6A) | ELECTRIC (120VAC 60HZ 0.6A) | ELECTRIC (110VAC 60HZ 0.5A) |
| POWER CONTROL | N/A | N/A | FLEXIBLE HAND CONTROL | POWER LEVER | POWER LEVER |
| CUSHION | SLOW RELEASE MEMORY FOAM | POLYURETHANE | MULTI-DENSITY WITH MEMORY FOAM CORE | POLYURETHANE | POLYURETHANE |
| CARRY HANDLE | EASY GRIP | EASY GRIP | ERGONOMIC DESIGN | EASY GRIP | N/A |
| COVER | MEDICAL GRADE, WATERPROOF | 100% POLYESTER | MEDICAL GRADE, ZIPPERED, WATERPROOF | 100% POLYESTER | 100% POLYESTER (4 COLORS) |
| WEIGHT CAPACITY | 80-230lb / 36-104kg 195-350lb / 88-159kg | 95-220lb / 43-100kg 200-340lb / 91-154kg | 300lb / 136kg | 300lb / 136kg | 300lb / 136kg |
| WEIGHT | 7.9lb / 3.6kg | 7.9lb / 3.6kg | 13.6lb / 6.2kg 16.9 lb / 7.7 kg | 12.3lb / 5.6kg | 64.6lb / 29.3kg |
| SIZE (DxWxH) | 19inx17inx4in 48cmx44cmx10cm | 19inx17inx4in 48cmx44cmx10cm | 19.6inx17.4inx4.3in/50cmx44cmx11cm 19.6inx19.7inx4.3in/50cmx50cmx11cm | 19.5inx17.1inx3in 50cmx43cmx8cm | 28.5inx30.7inx42.5in 72cmx78cmx108cm |
| WARRANTY | 1 YEAR LIMITED | 1 YEAR LIMITED | 1 YEAR LIMITED | 1 YEAR LIMITED | LIMITED LIFETIME* 2 YEARS: Motor & electrical parts. |
| HCPCS | E0629 | E0629 | E0628 | E0628 | E0627 |

*Metal support structure & chair (excluding fabric & foam).

ARE LIFTING SEATS THE RIGHT SOLUTION FOR YOU?

Do you struggle to stand from a seated position or need help to stand?

YES



Can you walk on your own or by using a cane or a walker?

YES



Are you active around your home and/or your community?

YES



THEN A LIFTING SEAT CAN WORK FOR YOU.

