

 **Posey®** FALL MANAGEMENT Posey® Door Guard/Stop Guard & Do Not Enter Sign/Banner

The Posey Door Guard/Stop Guard and Do Not Enter Sign/Do Not Enter Banner provide a strong visual reminder to residents not to exit through the doorway, or enter "off limit" rooms. The Door Guard (8210) and Do Not Enter Banner (7998) attach with hook and loop adhesive strips to the doorframe and remove easily in case of emergency. Both Guards fit doorframes up to 53" (135 cm) wide. The Stop Guard (8209) and Do Not Enter Sign (7997) attach with hook and loop adhesive strips directly onto the door.

- REF 7997** Do Not Enter Sign (sign only) **REF 8209** Stop Guard (sign only)
REF 7998 Do Not Enter Banner (sign and banner) **REF 8210** Door Guard (sign and guard)

Inspection

WARNING Inspect before each use. Check for broken stitches, or torn, cut or frayed material. Banner does not prevent residents from exiting through door.

Application Instructions:

- 7997** and **8209** 1. Remove backing from adhesive strips and apply to the front of the door.
7998 and **8210** 1. Remove backing from adhesive strips and apply strip vertically to each side of the doorjamb at approximately waist level.
2. Attach Door Guard/Do Not Enter Banner to each side.


NOTE: Replacement adhesive strips available at hardware/craft stores.

Disposal

WARNING Dispose of per your facility policy or applicable local code for BIOHAZARDOUS material.

Storage Instructions

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

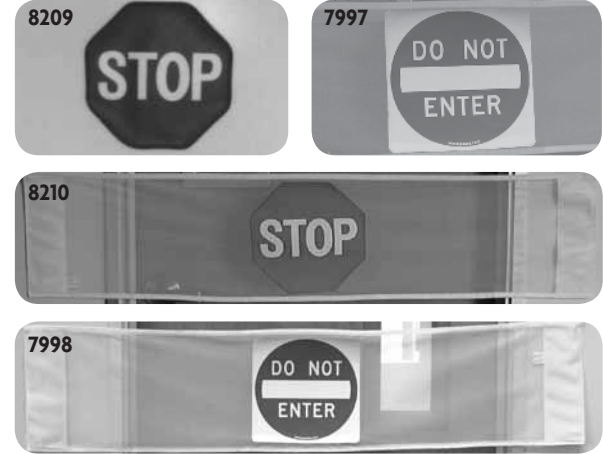

Washing Instructions:

7997 and **8209** - Wipe with mild soap and water or disinfectant.

7998 and **8210** - Machine wash warm, tumble dry low.

