



**Posey**®  
THERAPY PRODUCTS



Posey® ADL Aids

*Designed for patients/residents with diminished cognitive function. Bright colors and texture variation provide visual and tactile stimulation; threading of shoestring and buttoning buttons aid improvement of fine motor skills. Center plastic pocket holds photo of family or friend. Machine washable. One per package.*

**REF 7400** Activity Apron - Secures around waist

**REF 7410** Happy Hands Overlay - Attaches to Tray or Lap-Hugger

## Monitoring

**⚠ WARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

## Inspection

**⚠ WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

## Application Instructions:

**7400:** Attach Apron around the patient's waist using the ties. Make sure the Apron is snug, but does not interfere with breathing.

- 7410:**
1. Place the provided hook and loop adhesive strips on the under side of the Tray or Lap Hugger.
  2. Adhere the rough hook side of the Happy Hands Overlay to the loop side of the strip.



## Choking Hazard

**⚠ WARNING** Patients may attempt to remove buttons and put them in their mouth. DO NOT leave patients at risk of choking alone or unsupervised with this product.

## Disposal

**⚠ WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## Storage And Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

## Washing Instructions:



 Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • [www.posey.com](http://www.posey.com)  
Phone: 1.800.447.6739 or 1.626.443.3143 • Fax: 1.800.767.3933 or 1.626.443.5014