


Getting Started

#1 Locate push-button on Patch.



#2 Push and hold Patch button until the red light flashes.

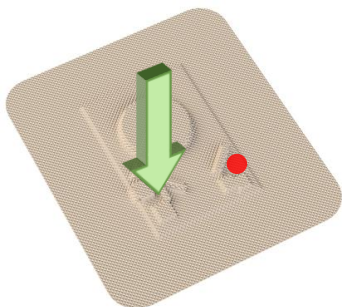


#3 Press RESET to turn-on Wedge. The **OUT OF RANGE** indicator will flash.



#4 Watch SIGNAL arc flash, indicating that the Wedge & Patch are **paired!**



#5 Remove adhesive backing from Patch perimeter & **place** it on the **back** of the body in one of areas shown.

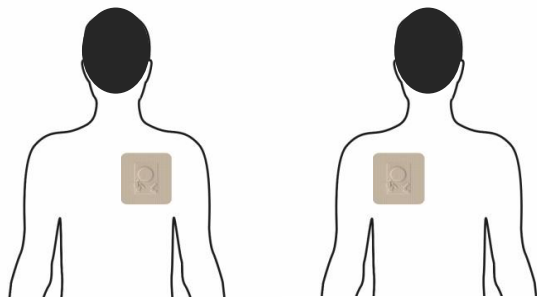

Changing the Patch

#1 Patch Change is required when the Wedge chirps and the **CHANGE PATCH** light flashes.



#2 Press **RESET** on the Wedge. The **OUT OF RANGE**, **CHANGE PATCH**, and **BATTERY LEVEL** lights will flash. The Wedge is now ready to pair with a Patch.



#3 Follow the steps on the **REVERSE** side of this card: **turn-on** the Patch, **pair** the Patch and Wedge; place the Patch on the **back** of the body as shown.

