

Washing / Drying Instructions

Washing

Washing is recommended before using Quik-Sorb™ Underpads.

- When washing, set on normal cycle using hot or warm water.
- If necessary, bleach may be used in a moderate amount. Excessive bleach will significantly reduce the life expectancy of the underpad.
- Softeners should be used in moderation. Excessive use of softeners will reduce absorbency by up to 25%.

Drying

- Maximum drying temperature is 160° F. Excessive heat will cause shrinkage and product breakdown.
- Allow a cool down cycle of at least five (5) minutes after drying cycle is complete.