



SONIC RING ELITE TELEPHONE / VIDEO PHONE SIGNALER MODEL TR75VR

SONIC ALERT'S LIMITED WARRANTY

The Telephone Ring Signaler Model TR75VR is warranted against manufacturing defects in materials and workmanship for five (5) years from the date of purchase. Within this period Sonic Alert will repair or replace at our option the TR75VR without charge for parts and labor. Simply send the TR75VR (postpaid) and a copy of your sales slip as proof of purchase to: Sonic Alert Inc., Attn: Repairs 1050 E. Maple Road, Troy MI 48083

FCC INFORMATION

We designed your telephone signaler to conform to federal regulations, and you can connect it to most phone lines. However, each TR75VR (and other device, such as a phone or answering machine) you connect to the phone line draws power. We refer to this power draw as the *ringer equivalence number*, or REN. The REN is on the TR75VR rating label.

If you use more than one phone or other device on the phone lines, add up all the REN's. If the total is more than five, Phones might not ring. In rural areas, a total of three might impair ringer operation. If ringer operation is impaired, disconnect a device from the line.

Your TR75VR complies with Part 68 of FCC Rules. You must, upon request, provide the FCC registration number and the REN to your local phone company. Both numbers are shown on the TR75VR's rating label.

In the unlikely event that your TR75VR causes problems on the phone line, the phone company can temporarily discontinue service. If this happens, the phone company notifies you of these changes in advance. If advance notice is not practical, the phone company notifies you as soon as possible and advises you of your right to file a complaint with the FCC.

Also, the phone company can make changes to its lines, equipment, or procedures that could affect the operation of this device. The phone company notifies you of these changes in advance, so you can take the necessary steps to prevent interruption of your phone service.

Modifying or tampering with your TR75VR's internal components can cause a malfunction and invalidate the TR75VR's warranty and void your FCC authorization to operation it. If the TR75 is not operating as it should, return it to Sonic Alert. If the problem is harming the phone lines, the phone company might ask that you disconnect the TR75VR until the problem is resolved.

OPERATION


The model TR75VR telephone / video phone signaler alerts you that your TTY, telephone, fax or videophone is ringing by flashing a lamp on and off automatically. The TR75VR can flash lights in other rooms by use of remote receivers, (Models SA101, SA201, RH100, BL300, and Sonic Boom Alarm Clock)

INSTALLATION

1. Plug the end of the TR75VR phone cord into the telephone input jack at rear of videophone (see videophone directions for telephone flasher unit). For telephone signaling plug the modular phone plug into standard modular phone jack (use jack type USOC RJ11C). The fifteen-foot cord is long enough to accommodate a variety of phone locations.
2. To set ringer flash pattern, move the three position switch on the side of the TR75VR marker 123 to desired flash pattern. If using an additional Sonic Alert phone signaler for your regular phone or TTY, this feature will allow you to select a unique flash pattern different from your other phone signaler.
3. Plug a lamp into the pluglet on the TR75VR. **Make sure the switch on the lamp is in the "ON" position or the lamp will not flash when the telephone rings.** The switch light will go on each time the lamp flashes. If the switch light comes on and the lamp is not flashing, make sure the lamp is in the on position and the bulb is not burned out.
4. Plug the TR75VR into any standard electrical outlet. **Do not use power strips or surge protectors these devices will reduce the strength of the signal being sent to remote receivers. If other receivers do not receive the signal from the TR75VR plug the TR75VR into an outlet by itself, other electrical appliances like TV's etc. can weaken the signal sent from the TR75VR if plugged into the same outlet.**
5. To use Remote Receivers (Model SA101, SA201, RH100, BL300 or SB1000) simply plug receiver into an electrical outlet and plug a lamp into the outlet located on the receiver. Models SA101, SA201, RH100, BL300 and the Sonic Boom Alarm Clock can be used.
6. Use with a video phone, plug in TR75VR into 110 volt outlet. Plug phone line into back of video phone at telephone input

The TR75VR and Remote Receivers are designed for incandescent light bulbs up to 300 watts. Never use the TR75 with a fluorescent light or any type of appliance.

Diagram TR75VR

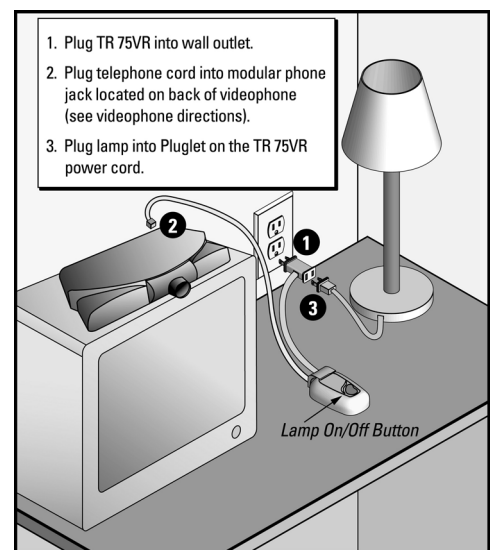

MADE IN CHINA

1050 East Maple Rd, Troy MI 48083 (248)577-5400
www.SonicAlert.com