




TURNWEDGETM

PLACEMENT INSTRUCTIONS

- 1.) Place a drawsheet or a folded sheet under the patient's back.
- 2.) Use the drawsheet to pull the patient on their side.
- 3.) Place the Turnwedge behind the patient's back, between the shoulders and the lower back.
- 4.) In the case of a patient with heavy drainage, you can insert a chuck into the folded drawsheet. This will provide you with the needed extra layer of absorbency.
- 5.) In the event that the TurnWedge becomes soiled, it is designed to be easily wiped off with any patient-friendly cleaning solution and a damp towel.

*Note: **DO NOT** put the Turnwedge inside a pillow case. Doing so will cause the Turnwedge to lose its non-skid surface. Always place the Turnwedge between the drawsheet and the sheet covering the mattress.*

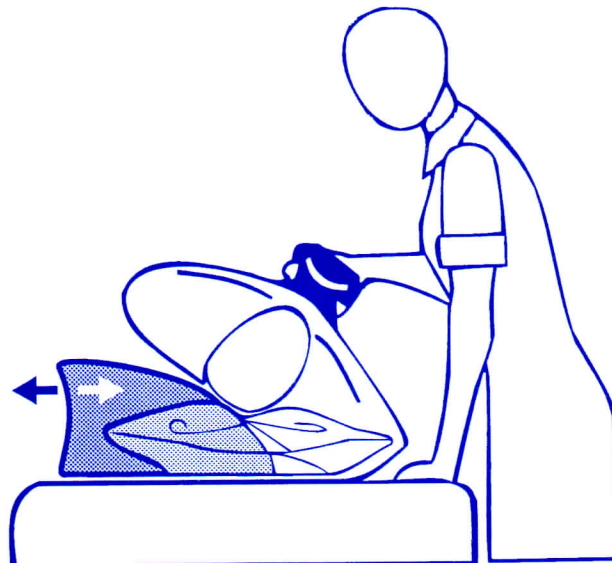

For more information, please call CarePlus Medical, Inc. at 888-467-7607

or visit us at www.turnwedge.com

Patent #D531315

MADE IN THE U.S.A.