


# Small O2 Tank Holder Mounting Instructions

Measure Seatback Before Mounting To Determine Strap Setting.

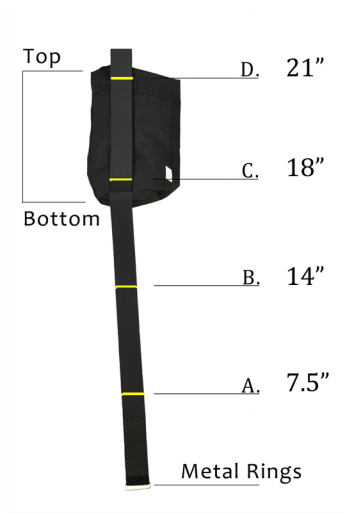

1. Bag as shipped from Manufacturer has a **factory setting for a medium seatback**. Measure seatback first - If your seatback is smaller or larger than approx. 14 inches, refer to 'Strap Settings' below.

2. Undo the top velcro connection, releasing the metal ring. Bring metal ring down, facing the bottom of the bag. **Note:** Markers A-D are not lines, and are marked by white dots along the sides of the strap.

3. Position top Velcro connections at the top of the seatback and hold. Thread metal ring under seatback and bring up the backrest.

4. Meet metal ring and the bag at the top of the seatback. Feed top Velcro attachment through the metal ring.

5. Secure the Velcro so that bag is stable. Slip resistant grip will help prevent strap from sliding.

## Strap Settings

**Note:** Strap settings are just a guide. Additional adjusting may be needed.

### Small Seatback

For a seatback about 71/2 inches set straps with position 'A' just above the O2 Tank loop closest to the top.

### Med. Seatbacks

For a seatback about 14 inches, set straps with position 'B' just above the O2 Tank loop closest to the bottom.

**Factory Setting**

### Large Seatbacks

For a seatback about 18 inches, set straps with position 'C' just above the O2 Tank loop closest to the bottom.

### XL Seatbacks

For a seatback about 21 inches, set straps with position 'D' just above the O2 Tank loop closest to the bottom.



**DIESTCO**  
**MANUFACTURING**  
**CORPORATION**



*Proudly made in the  
U.S.A.*

We strongly encourage you to call us at (800) 795-2392 if you have any problems, questions, or comments. We are always developing new products for wheelchair, scooter, crutch, and walker users... So let us know if there's something special you need!