

Parts Reference - Bariatric Shower Chair (Item #72621)



72621



Part A

Part B & Part C



Part D



Part E

Assembly Instructions

1. Take one leg frame (Part B or Part C, it does not matter which), and lay it flat on the ground.



2. Take the seat with the two pre-assembled tubes located under the seat (Part A), and insert the two tubes into the leg frame (Part B or C). Make sure the brass buttons snap firmly into the leg frame.



3. Take the other leg frame (Part B or Part C), and insert the seat tubes on the opposite side of the seat (Part A) into the leg frame. Make sure the brass buttons snap firmly into the leg frame.



4. Take the 4 legs (Part D), and insert them into the bottom of the leg frames. Set the chair height by using the same button-hole location for each leg. Make sure the brass buttons snap firmly into the leg frame. For your safety, adjust the rubber tips on the legs by turning them so they make maximum floor contact.



5. Take the backrest (Part E), and insert it into the two oblong backrest holes located on top of the seat towards the back side of the seat. Push the backrest down into the backrest holes, snapping it firmly into place.

