



LaserCane™ Instructions

Adjusting the Height

1. Loosen the tightening ring in the middle of the cane by turning it.
2. Push in the height adjustment button and pull apart the two sections of cane until you find the correct height for you. The cane comes set at the lowest height. Please do not try to make it smaller or you may damage the wiring inside the cane.
3. Re-tighten the tightening ring.

Operating the LaserCane

1. When you receive your cane, the red on/off button located in the middle of the cane is off. Push in this button until it clicks on.
2. Now, when you press the cane down on the floor, a red line should be projected on the floor.
3. If not, click the on/off button again and then repeat Step 2.

Walking With the LaserCane

1. When you are experiencing hesitation, put the cane in front of you as you would use a cane and try to step over the red line on the floor.
2. If you are not freezing and want to use the cane, turn off the laser by pressing the red on/off button.

Replacing the Batteries

The LaserCane is powered by two AA batteries located in the body of the cane.

1. Loosen the tightening ring.
2. Push in the height adjustment button and separate the two tubes approximately eight inches. Remove the two batteries and replace with a fresh set of alkaline batteries (important for electronic products).
3. Carefully put the cane back together and reset the height.

