

# I N S T R U C T I O N S



## InBed™ Head Wash System

Catalog Number 76427-1000

**Note:** Basin drain is located inside the basin of the InBed™ Head Wash System. Drain spout is located on one of the side walls.



- 1) Before use, open basin drain and loosen drain spout for ventilation.
- 2) Position the person's head, neck and shoulders on the neck support.
- 3) Moisten hair and shampoo.
- 4) Rinse and towel dry hair thoroughly.
- 5) Before closing the basin drain make sure all the water has drained into the unit.
- 6) Close basin drain and drain spout for transport.
- 7) To empty, stand the unit upright, loosen basin drain and open drain spout.
- 8) Empty water out of the drain spout into a sink.

**Note:** In order to prevent leakage make sure the two neoprene gaskets are always in place when closing the basin drain.

**Note:** Unit should be cleaned using a mild soap or detergent if desired. A spray disinfectant can be used to clean the device and prevent mildew. Make sure to thoroughly rinse after using any disinfectants or detergents.

## Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA  
973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)