




Universal Electric Razor Holder

Directions

First, insert the body of the razor into the horizontal Velcro strap which is designed to encircle the razor. The razor head should protrude above the strap at a convenient height for shaving when the handle is placed around your hand. Tight the horizontal Velcro strap by threading it through the D-ring and back on itself.

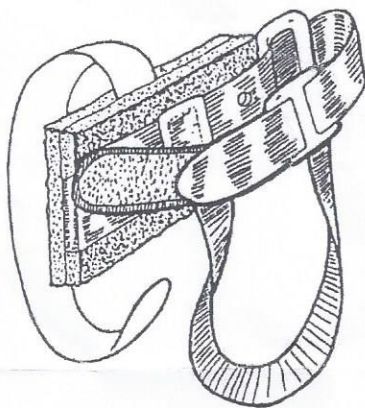

The vertical Velcro strap wraps around the base of the razor to keep it from sliding down. The vertical strap can be shortened/ lengthened by keeping it taut against the razor and pressing it against the outside of the horizontal strap.

The handle of the holder is made of soft steel which can be bent to fit your hand.

Examples:

