

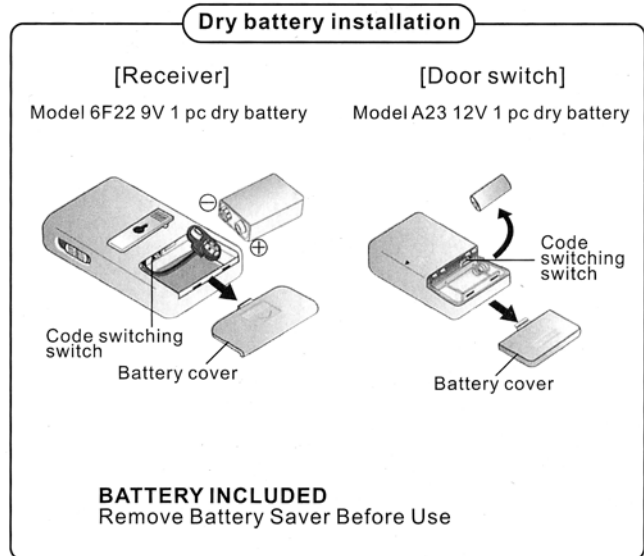


Operating instruction

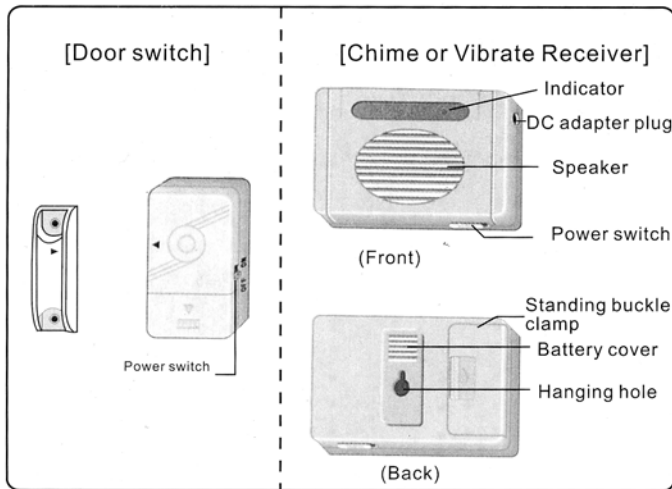


Instruction

1. Correctly install battery on the door switch by according to specification instruction, and then turn on power switch, which is at the side of the door switch.
2. Correctly install battery on the main body of receiver according to specification instruction, and then set power switch to Chime or Vibrate position
3. Confirm if receiver could let out chime when the distance of magnet and door is more than 1 cm.
4. Install magnet on the door by dual-face adhesive tape or screw supplied by the product.
5. As is shown in the picture, put magnet parallel with door switch on the wall or other places.
6. Confirm if receiver let out chime when it is turned on.

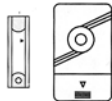

•Replace Battery When Low Battery Indicator Lights.

Operating range for door switch

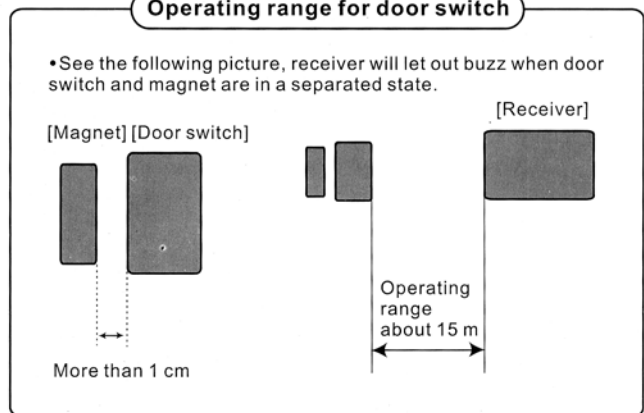

Specification

Battery	Receiver: Mode1 6F22 9V 1pc dry battery (unsupplied) Door switch: Mode1 A23 12V 1pc dry battery (unsupplied)
Life of battery	Receiver: about 90 days (alkalescency dry battery) Door switch: about 90 days (alkalescency dry battery)
Specification	Receiver: 111(w)*71(H)*34(D) mm Door switch: 35(w)*60(H)*19(D) mm
Volume of receiver	High: about 85 dB Low: about 80 dB

WANDER ALARM

For Doors or Windows

Easy to Install and Use
Sending Unit Can be Mounted with Screws or Double Sided Tape

This Product is not for use in life threatening or safety situation-

PRODUCT #5959