Español

SILOPAD™ Gel Tubular Gel Tubing

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WMDE
Bergerweg 18
6085 AT Horn
Holland

www.silipos.com

Made in U.S.A.
Licensed under U.S. Patent number
6,552,109

9576-8

SILOPAD™ Gel Tubing

GEL INSIDE

English

SILOPAD™ Gel Tubing

- A soft, comfortable and stretchable fabric fully coated with our proprietary polymer gel which slowly releases a medical grade mineral oil (USP) to soften and moisturize the skin.
- Product is dermatologist tested, washable and reusable.

Indications:

- Helps relieve pressure, friction, and shear forces on the affected digit.
- Product #'s 10855, 10865 and 10875: for digit protection and moisturization.
- Product # 10845: for arm/leg protection and moisturization.
- Also helps to reduce scar tissue.

Directions for Use:

- Wash and dry affected area before applying product.
- Product may be cut to desired length (elastic will not unravel).
- Slip product over toe or finger (arm or leg for Product # 10845) and place the gel directly over the affected area.
- Remove product at least 3 to 4 hours daily to allow skin to breathe.

Recommended Care:

We recommend that you hand wash product. Air dry product and lightly dust product with talcum powder after drying.

WARNING: Do not place product on an open wound. If irritation, discomfort or poor circulation occurs, discontinue use and notify your physician immediately. KEEP OUT OF REACH OF CHILDREN.

Product #10845	Mesh Tubing Wide ~3" x 10" (~7.6cm x 25cm) 1 per pkg.
Product #10855	Mesh Tubing Narrow ~3/4" x 6" (~2cm x 15cm) 3 per pkg.
Product #10865	Digitube Plus Narrow ~3/4" x 6" (~2cm x 15cm) 2 per pkg.
Product #10875	Digitube Plus Wide ~1" x 6" (~2.5cm x 15cm) 2 per pkg.

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