## Safe-t mate

## Product use and specification sheet

Model Number	SM-014
Brand Name	Safe•t mate ®
Description	Wheelchair Anti-rollback Device Specific to Invacare Tracer EX2
	Also fits Invacare Tracer SX5
Manufacturer	210 Innovations, LLC
	210 Leonard Drive
	Groton, CT 06340 860.445.0210
Recommended Use	The Safe•t mate Wheelchair Anti-rollback Device is intended as a supplemental
	attachment and not a replacement for factory installed hand brakes. It is
	recommended for wheelchair users with poor safety awareness who forget to set
	the manual hand brakes before exiting or entering the wheelchair.
	This device may also be appropriate for use with higher functioning individuals to
	assist in activities of daily living.
Not recommended for	Patients who require a physical restraint.
Not recommended for	Tationto who require a physical restraint.
Operation	Anti-rollback Device
	The device consists of left and right mounting bracket / actuators arms. When
	properly installed, the actuators depress under the weight of the occupied
	wheelchair seat. This in turn rotates the brake arms off of the tires. In this state
	the wheelchair moves freely in all directions. The brake arms are biased toward
	engaging the tires (through spring tension) as weight is lifted from the seat. Once
	engaged, the brake arms prevent the wheels from rotating backward. The chair is
	still able to move forward in the engaged position. The unit features temporary
	override handles to assist caregivers in moving the unoccupied chair backward.
	This device allows the chair to be quickly folded for transport or storage.
	This device can be used with seat cushions.
	This device is not compatible with solid drop seats.
Specifications	Materials used
	Chrome plated carbon steel, Stainless steel, Glass filled polycarbonate plastic.
	Does not contain Latex.
	Application
	SM-014 will fit all available configuration of the Invacare Tracer EX2 or SX5
	This includes 16",18" or 20" chair widths, permanent or removable arms and
	either axle height.
Care	All Anti-rollback Device components are safe to remain on chair during
	wheelchair washing.

Word Doc SM-014 Prod use spec sheet