

# Easy Pushbar Assembly Instructions

MADE IN CHINA


## Parts List

A	Pushbar
B	Clamps
C	Thumb Screws
D	Metal L Bar
E	Turn Knob


Notice: Warranty in effect only if (4) Thumb screws (C) and (2) Turn-knobs (E) are hand tightened for proper functioning

### Step 1

Attaching Handle Clamp (B) to Metal L Bar (D)

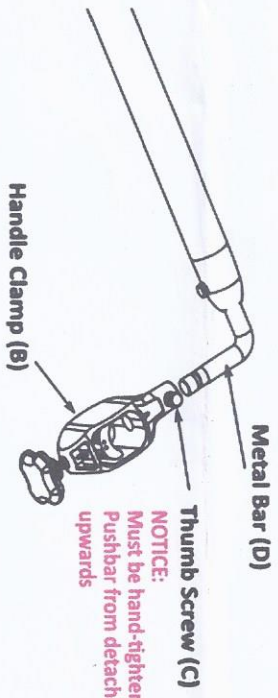


1. Insert the handle clamp (B) into the metal L bar (D) as shown in the diagram.
2. Push the bar as far into the handle clamp as possible.
3. Lock the assembly by hand tightening the thumb-screw (C).
4. Repeat for the second handle clamp.

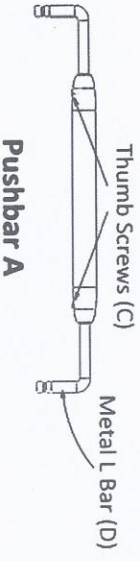

### Step 2

Attaching Pushbar (A) to wheelchair hand grips (F)

Thumb screws (C)

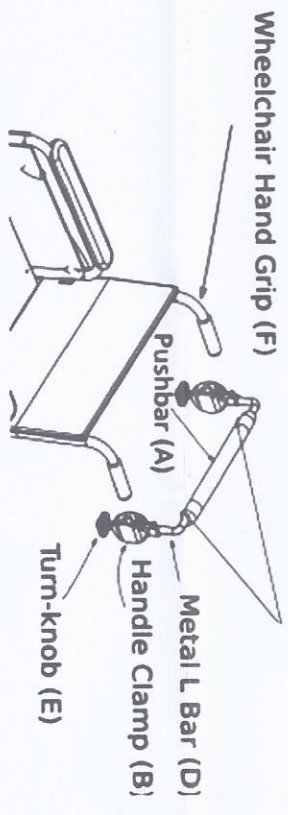

1. Slightly loosen the thumb screws (C) on the pushbar (A) so that the metal L bar (D) slides easily in and out.
2. Attach one handle clamp (B) to one hand grip (F) of the wheelchair.
3. Adjust the width of the pushbar to fit the wheelchair by sliding the metal L bars (D) in and out of the pushbar (A).
4. Attach the other handle clamp (B) to the other wheelchair hand grip (F).
5. Tighten both handle clamps using the Turn-knobs (E).
6. Center the pushbar (A) and then lock in place by hand tightening the two thumb screws (C) on the pushbar.

