
EasyPushbar Assembly Instructions Security Version

MADE IN CHINA Lock Screws (C)

Pushbar Allen Wrench Clamp Bolt Metal L Bar **Lock Screws** Clamps

Parts List

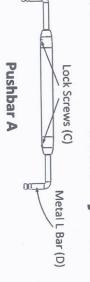


Allen Wrench (G)




Attaching PushBar (A) to wheelchair hand grips (F) Wheelchair Hand Grip (F)

Handle Clamps (B)

Lock Bolts (E)

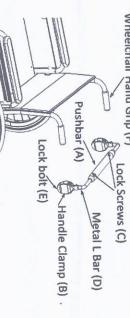

Attaching Handle Clamp (B) to Metal L Bar (D) Step 1

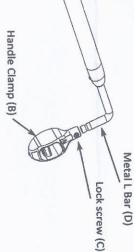

- 1. Insert the handle clamp (B) into the metal L bar (D) as shown in the diagram.
- Push the bar as far into the handle clamp as possible
- Lock the assembly by tightening the lock-screw (C) using the allen wrench provided.
- Repeat for the second handle clamp.

- 1. Slightly loosen the lock screws (C) on the pushbar (A) so that the metal L bar (D) slides easily in and out.
- 2. Attach one handle clamp (B) to one hand grip (F) of the wheelchair.
- Adjust the width of the pushbar to fit the wheelchair by sliding the metal L bars (D) in and out of the pushbar (A).
- Attach the other handle clamp (B) to the other wheelchair hand grip (F)
- 5. Tighten both handle clamps using the Clamp Screws (E) using the allen wrench provide
- Center the pushbar (A) and then lock in place by tightening the two lock screws (C) on the pushbar.