

# CARSON®

AM-20

## ATTACH-A-MAG™



FLEXIBLE CLAMP-ON LED LIGHTED MAGNIFIER

### Instructions for use:

#### To Activate LED Light



To turn light on, slide switch on top of LED housing unit up (Fig. 1).

**Note:** Please remove battery tab before first use.

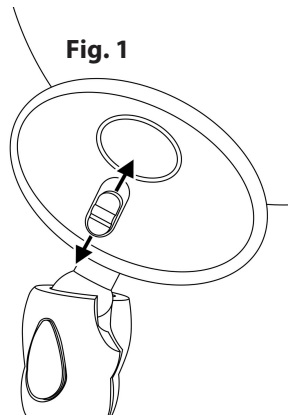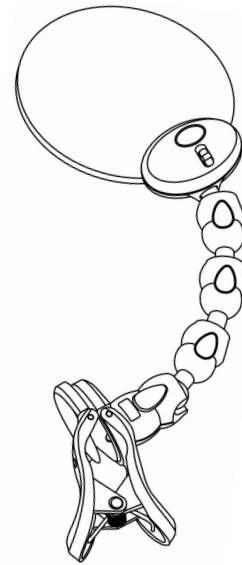




#### To Attach to Table

Squeeze to open base clamp (Fig. 2). Clamp onto table edge as shown (Fig. 3).

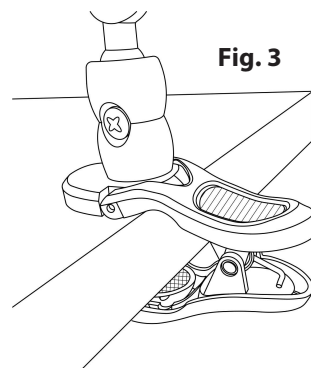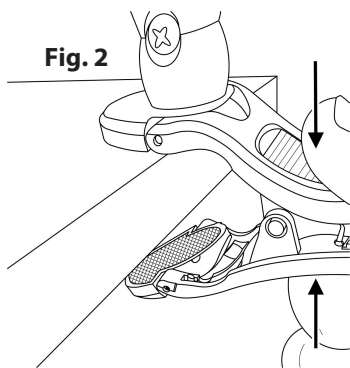





#### Position Magnifier Lens

Bend and twist flexible arm to position lens as needed (Fig. 4). If flexible arm is too loose or too tight it can be adjusted. Use a phillips head screw driver to loosen or tighten screws along flexible arm to adjust tension (Fig. 5). If desired, segments of flexible arm can be removed by loosening screws and pulling segments apart (Fig. 6). Attach remaining segments and tighten to secure.

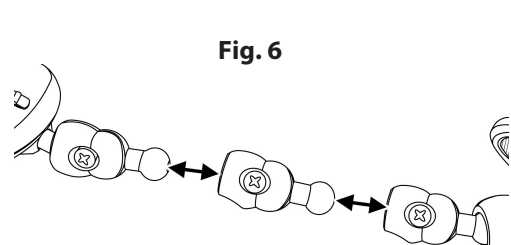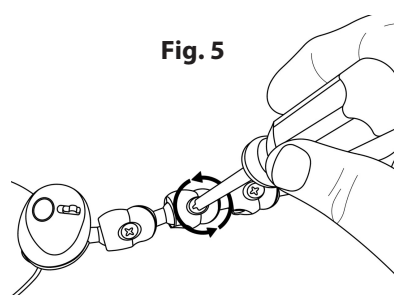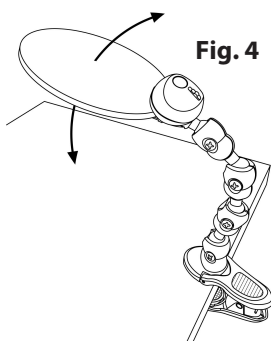

## Replacing Batteries

Carefully pull back LED housing unit and remove from magnifier neck (Fig. 7). Remove safety screw and pull back battery door (Fig. 8-9). Remove spent batteries and replace with three fresh G12 button cell batteries according to polarity indicators marked on battery door (Fig. 10). Replace battery door and snap LED housing unit back onto magnifier neck.

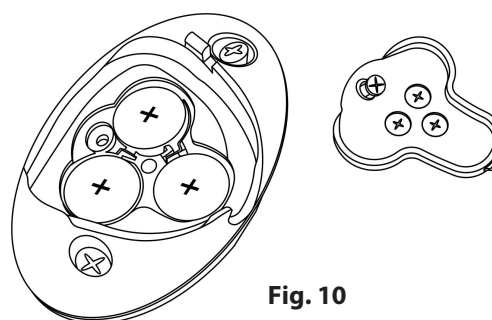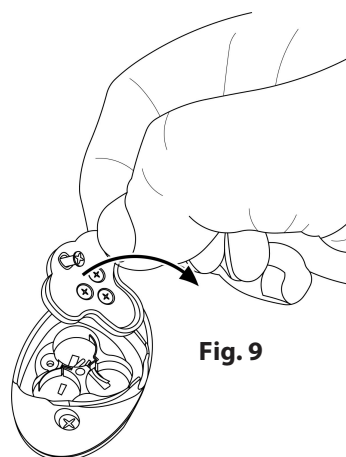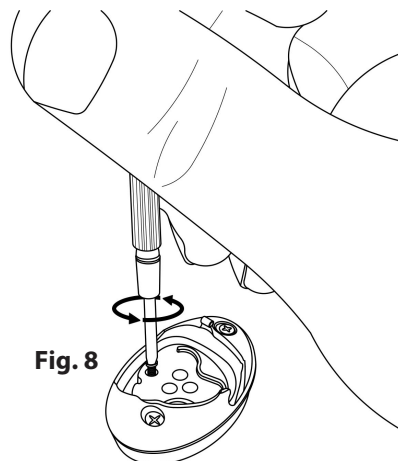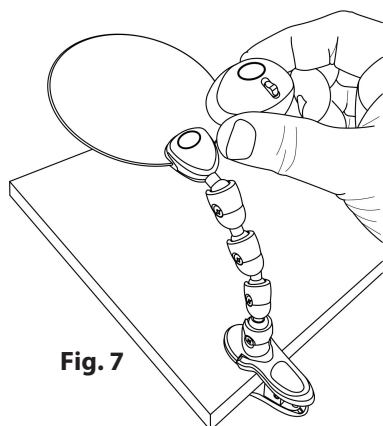

**Warning:** Use only alkaline batteries. Mixing old and new, different brands or types within brands may cause batteries to leak, overheat or explode. Keep away from fire or excessive heat. Inspect batteries for corrosion or leakage. Insert in direction indicated. Rechargeable batteries not recommended.

## Cleaning Your Magnifier

Clean lens with a soft, clean microfiber cloth by rubbing gently in a circular motion. Moisten the lens cloth with water if needed. Do not use harsh chemicals, solvents or abrasives as they may damage the lens.

## CAUTION - FIRE HAZARD:

Keep magnifier away from direct sunlight to prevent accidental fire.

### Customer Service:

If you experience any difficulties, please contact us and we will be happy to help you.

#### In the U.S.A.

**Phone:** 1-631-963-5000

**Email:** info@carson.com

#### In the U.K.

**Phone:** +44 (0) 1938 557440

**Email:** uksupport@carson.com

#### In the E.U.

**Phone:** +44 (0) 1938 557440

**Email:** eusupport@carson.com

**For warranty information, visit [www.carson.com/warranty](http://www.carson.com/warranty)**