



Etac Molift Raiser

Transfer platform for active transfer

etac®

Etac Molift Raiser – for active transfer

Etac Molift Raiser sit to stand transfer platform is designed to safely activate the user whilst maintaining the best possible ergonomic moving and handling conditions for the caregiver.

Designed for both users and caregivers.

The Molift Raiser sit to stand transfer platform makes it fast and easy to prepare a transfer. The unique leg support is designed to be easily adjusted with one hand and can be controlled from a standing position, rather than kneeling. Similarly, the foot brake (right or left) will activate both brakes simultaneously providing a safe braking solution in space limited environments. The profile of the platform allows it be moved very close to the user, to allow the best possible involvement of the user in the transfer. The fine grain finish of the foot-plate provides a non-slip surface.

The Molift Raiser offers a safe and comfortable transfer. The padding in the leg support improves stability and the handle offers many different grip possibilities. These features enable the user to feel safe throughout the entire transfer.

The Molift Raiser is stable and can turn within its own radius which makes it easy to maneuver.



Name

Molift Raiser 16090202

Accessories

Soft Handles, 2-pack	82511
Raiser Strap S/M	82508
Raiser Strap L/XL	82509
Raiser Strap+ S/M	82528
Raiser Strap+ L/XL	82529
Raiser Belt S/M	82964 (82508 + 82963)
Raiser Belt L/XL	82965 (82509 + 82963)

Safe Working Load

150 kg / 330 lbs

Total Weight

14.2 kg / 31.3 lbs

Materials

Steel, plastic, aluminium, TPR



Features



Leg support with one-hand height regulation

The soft padded leg support provides comfortable support for the user. It is simple to adjust the height with only one hand.



Handle with many grip possibilities

The handle provides many grip possibilities for both caregivers and users.



One foot locks both wheels

The foot brake locks both sides whether you choose the left or right one. This ensures a smooth and safer use of the product.



Low boarding

The low step-on height makes it easier for users to place their feet on the platform.



A transfer platform with accessories that stimulate activity.

The Molift Raiser helps to activate the user thus offering the benefits of improving circulation, muscle activation and a resulting increase in self-confidence. The Molift Raiser can be used to assess the user's standing capability and helps encourage early mobilization.



A soft extended handle provides better grip for persons with reduced mobility.



A safety strap can provide support around the back or hips when standing up.



Accessories



Soft handles

If the user has difficulty grasping the handle and cannot reach all the way forward, the soft handle can be used to further help the user. Supplied in pairs.



Safety strap – Raiser Strap

If the user feels unsafe during a transfer, a safety strap can provide support around their back or hips when going from sit to stand and also whilst standing.



Safety strap with sliding function – Raiser Strap+

When users need help to be assisted, this can be done by one or two caregivers using the Raiser Strap+.



Raiser Belt - provides additional support

The Raiser Belt provides additional standing support. The belt can easily be placed around the buttocks or the waist depending on what best suits the user and the situation. Can be used by one or two carers.