



Gel Ovations™

Crutch Handle Cover

Gel for Comfort

stretches over existing pad
to add comfort to all
standard crutches.

www.gelovations.com

- Improves comfort and decreases pressure
- Highest quality products on the market
- Easy to install – no tools required
- Washable neoprene/nylon non-slip cover
- Latex free

Installation Instructions

This product is designed to fit right over the existing handle grip. Open the Gel Ovations™ Crutch Handle Cover by undoing the Velcro® so that the 'soft' loop side of the Velcro® faces up. Place between the support arms of the crutch on top of the hand grip. Place the center 'gel section' on top of the hand grip, and wrap the velcro loop side so that it faces down. Bring the 'rougher hook side' around, stretch tight, and press it closed against the loop to connect the two parts of the Velcro®. One small section at a time disconnect and straighten the seam until it is 'flat and tight'. For most comfort, use so that the seam faces the body and is only touched by the thumb.

Cleaning

Do NOT put pads/covers in a washing machine.

Do NOT submerge them in water or agitate.

Do NOT put in clothes dryer.

With HEATED DRYING turned OFF, you may wash them in the top rack of an automatic dishwasher.

Alternatively, wipe the surface with any normal household soap or cleaning solution that is not irritating and to which you are not allergic.

Limited Warranty

Gel Ovations™ warrants that this product will be free of manufacturing defects for a period of 12 months. If defective, please return to place of purchase for an exchange or full refund. This warranty does not extend to abuse of the product such as tampering or modification.

Gel
Ovations™

www.gelovations.com



Product Code: **CRTC**