

Leg & Foot Huggers provide leg support and help prevent resident's feet from slipping off the footrest and getting caught under the chair. Footrest extenders help to prevent footdrop by increasing the base of support and elevating the footrest. Cushioned with foam padding and covered in water-repellent, wipe-clean vinyl. Fits 18" to 20" (46 cm to 51 cm) wide wheelchairs. All models meet CA #117 flame retardancy standards.



- REF 6351** Foot Hugger – 1" (3 cm) Thick foam, 13"W x 9"D x 1"H (33 cm x 23 cm x 3 cm)
- REF 6352** Foot Hugger – 2" (5 cm) Thick foam, 13"W x 9"D x 2"H (33 cm x 23 cm x 5 cm)
- REF 6353** Foot Hugger – 3" (8 cm) Thick foam, 13"W x 9"D x 3"H (33 cm x 23 cm x 8 cm)
- REF 6354** Foot Hugger – 4" (10 cm) Thick foam, 13"W x 9"D x 4"H (33 cm x 23 cm x 10 cm)
- REF 6355** Leg Hugger – No footrest extender, 15½"W x 9"H (40 cm x 23 cm)



MONITORING

WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSPECTION

WARNING Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or buckles that do not hold securely. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS:

- WARNING** Patients should not attempt or be allowed to exit wheelchair unassisted without first removing this product from the wheelchair.
1. Position the patient in the wheelchair and adjust the footrests to fit.
 2. Release the quick-release buckles on the product attachment straps.
 3. Position the Hugger between the footrest bars with the extender resting on top of the footrests as shown in the photo.
 4. Wrap the top straps around the outside of each footrest hanger and fasten the quick-release buckles. Follow same procedure for the lower straps.
 5. Adjust the straps so the leg rest sits flush between the footrest bars. This will allow more room on the footrests for the resident's feet.
 6. Re-adjust footrest if necessary to compensate for extender cushion.

CLEANING INSTRUCTIONS

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



DISPOSAL

WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.