

Posey Smoker's Aprons cover a seated smoker and help prevent accidental burns to the skin and/or clothing. Hot ashes and lit cigarettes will not burn through the fabric, however, they may scorch the fabrics or leave a burn mark on the surface. These Smoker's Aprons can be used in a variety of applications with individuals in wheelchairs, geri-chairs, dining room chairs, etc. The aprons are reversible, with no front or back for extended life.

POSEY PREMIUM SMOKER'S APRON

Constructed from a high quality, breathable fabric woven from Nomex® and Kevlar® fibers and is inherently flame retardant. It is completely launderable and will always remain flame retardant throughout its life. Navy blue fabric.



REF 4551 Posey Premium Smoker's Apron, 30"W x 36"H (76 cm x 91 cm)

WASHING INSTRUCTIONS:

The Posey Premium Smoker's Apron can be laundered as follows: NON-CONTAMINATED aprons can be washed in lukewarm water (100°F / 38°C), without bleach, and dried under low heat in order to maintain color brightness and product longevity. Posey recommends washing CONTAMINATED aprons in hot water (160°F / 71°C) and cleaning per the "CDC Guidelines for Environmental Infection Control in Healthcare Facilities" (HICPAC). MMWR 2003; 52 (No. RR-10) page 27. This procedure may cause color to fade.

Non-Contaminated Aprons:



Contaminated Aprons:



POSEY ECONOMY SMOKER'S APRON

Constructed from fabric woven from fiberglass and finished on both sides with a Silicone based coating. It will remain flame retardant after repeated cleaning. Light gray fabric.



REF 4551E Posey Economy Smoker's Apron, 30"W x 36"H (76 cm x 91 cm)

CLEANING INSTRUCTIONS:

- DO NOT launder. Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



INSPECTION

WARNING Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or buckles that do not hold securely. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS (ALL APRONS):

1. Comfortably secure the apron around the resident's neck with the quick-release buckle and adjust the neck strap so that the top of the apron rides above the resident's chest and covers the individual's clothing.
2. Tie the upper set of securing straps behind the individual's shoulders.
3. Tie the lower set of straps around the individual's lower torso.
4. The straps can be trimmed in length if desired.

DISPOSAL

WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



4551



4551E

WARNING Make sure the patient cannot slide down or fall off the chair seat and become entrapped by the Smoker's Apron. If their body becomes suspended by the apron, suffocation could result.

WARNING This apron is not a substitute for proper supervision and all patients or residents should ALWAYS be closely supervised while smoking. Peripheral materials used in the construction of the apron such as the threading, straps and labels, represent less than 1% of the product. They DO NOT offer the same level of flame resistance and pose only a remote risk of fire hazard.